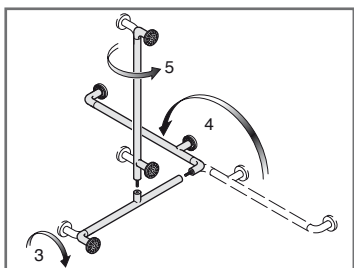
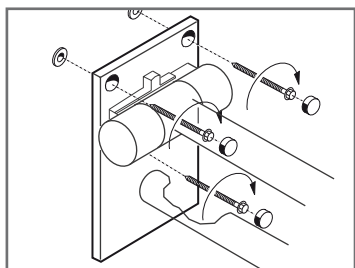
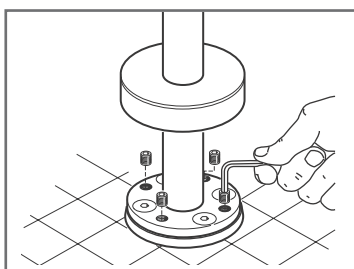
MANUALE DI ISTRUZIONI  
INSTRUCTION MANUAL  
NOTICE D' UTILISATION  
BENUTZERHANDBUCH  
MANUAL DE INSTRUCCIONES  
H A N D L E I D I N G

**PROGRAMMA  
400 - NY**  
**PROGRAMMA  
400 - SS**

**ITALIANO**

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**ENGLISH**

Index	Page
<b>PROGRAMMA 400-NY NYLON</b> Safety precautions Indications for maintenance and cleaning Installation procedures by standard fixing equipment Assembly of mirror lamps How to transform a safety support like 443/B-443/BL from left to right Fixing material Guide to project Technical explanations and certifications	<b>3</b>
<b>PROGRAMMA 400-SS STAINLESS STEEL AISI 316L</b> Safety precautions Indications for maintenance and cleaning Installation procedures by standard fixing equipment Assembly of mirror lamps How to transform a safety support like 443-SS/B-443-SS/BL from left to right Fixing material Guide to project Technical explanations and certifications	<b>33</b>

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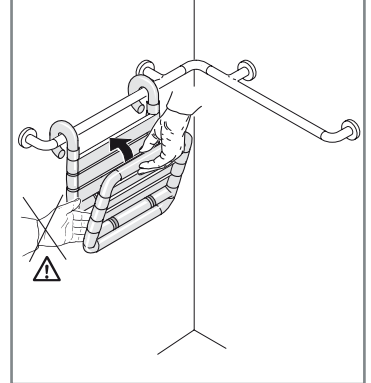
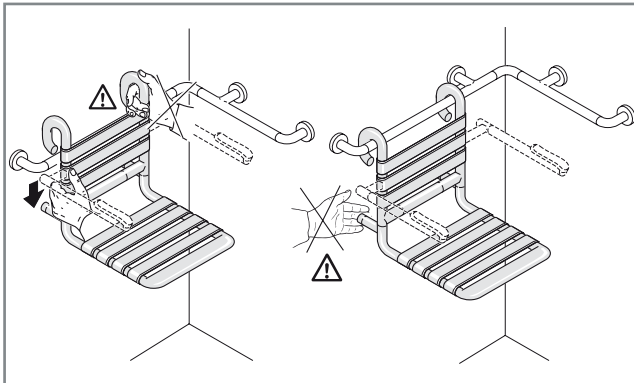
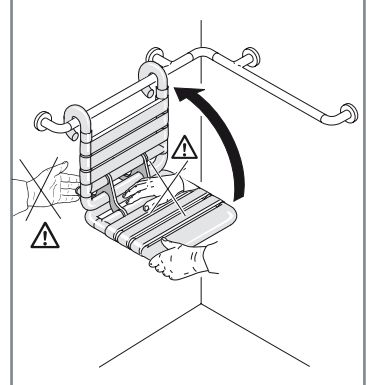
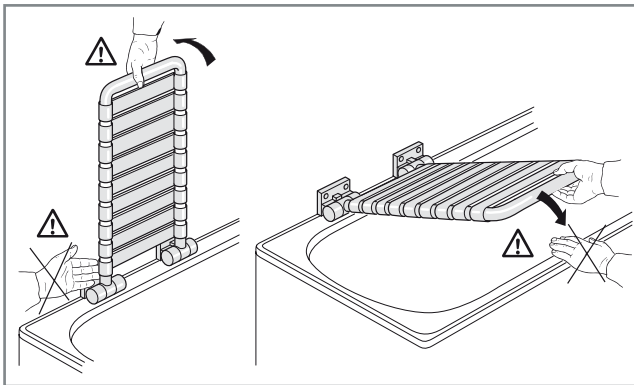
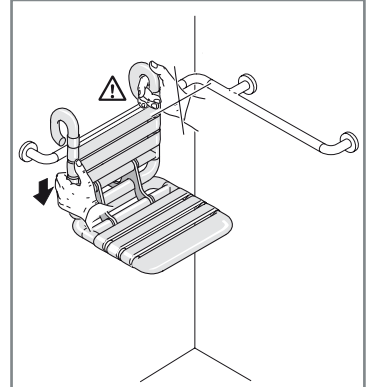
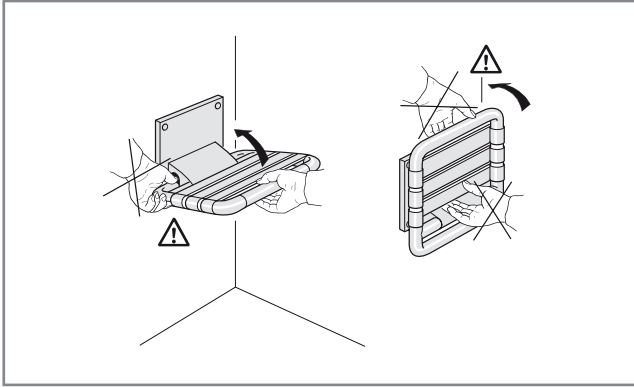
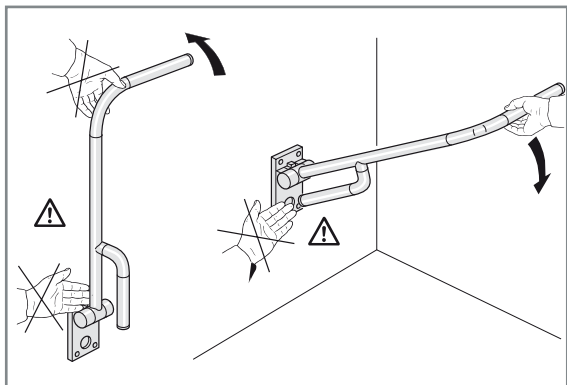
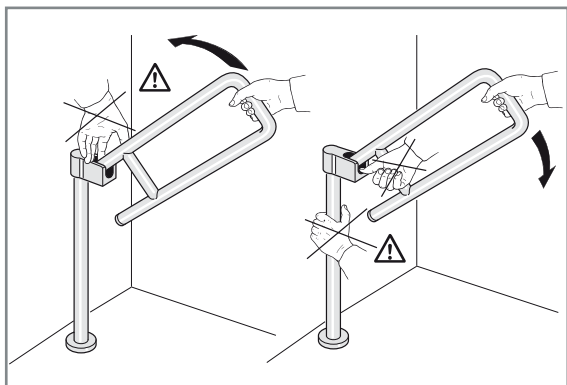
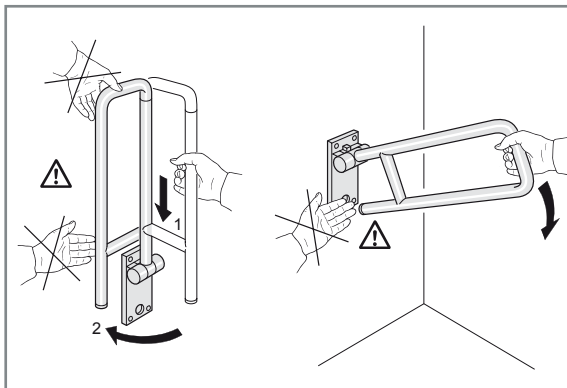
WARNING: All disabled accessories must be fixed in accordance with the table of fixing materials. PBA S.p.A. is not liable for any damage/loss of functionality of the product caused by not observing the recommendations set out in our instruction manual. Under NO circumstances must the PBA S.p.A. disabled and bathroom accessories be altered, tampered with or modified. Non-observance of these instructions makes product guarantees null and void.

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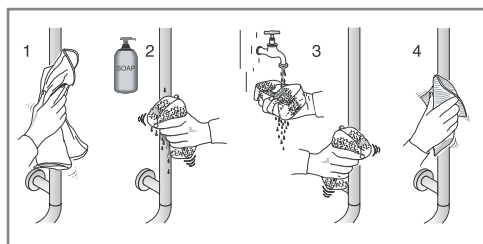
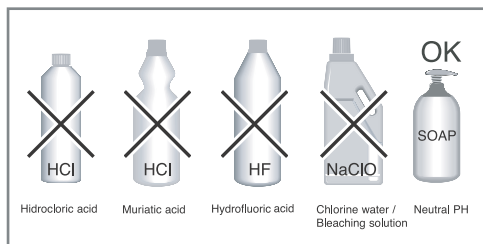
For cleaning PBA Nylon, please DO NOT use soaps or detergents containing chlorine, chloroform, chloride water, acids, bromine, cresol, carbolic acid, formaldehyde, formalin or potassium permanganate regardless of percentage content. PBA declines all responsibility for damage caused by use of above mentioned aggressive detergents. The nylon parts feature excellent stability to hydrocarbons, oil fats. They also have good stability to halogenated hydrocarbons such as carbon tetrachloride, frigen and freon. According to the above, we recommend disinfectants ba alcohols and also ammoniac and phosphor compounds.

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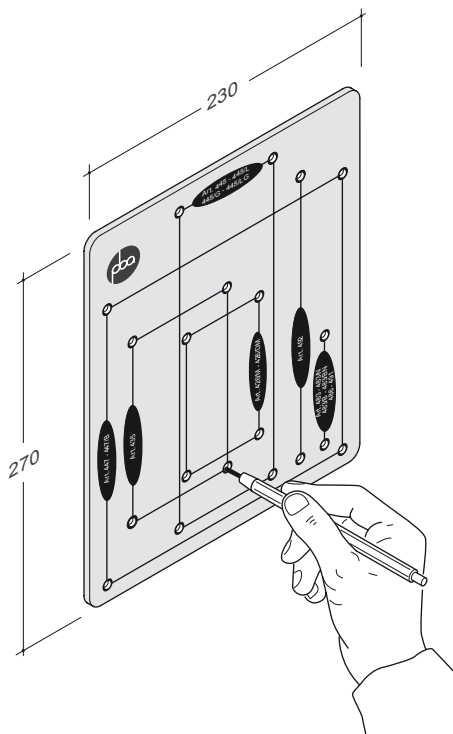
The following installation procedures refer to the fixing of disabled hardware to concrete. For other types of building construction, refer to the table of fixing material.

For articles 445, 445R, 441/P 447, 435, 426/M, 426/DM, 492, 483, 483/N, 483/B, 483/BN, 486, 491 we recommend the use of assembly template, art. 499, to set out the fixing hole centres.

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Lift-up seat for bath tub with plates 120x90 mm.

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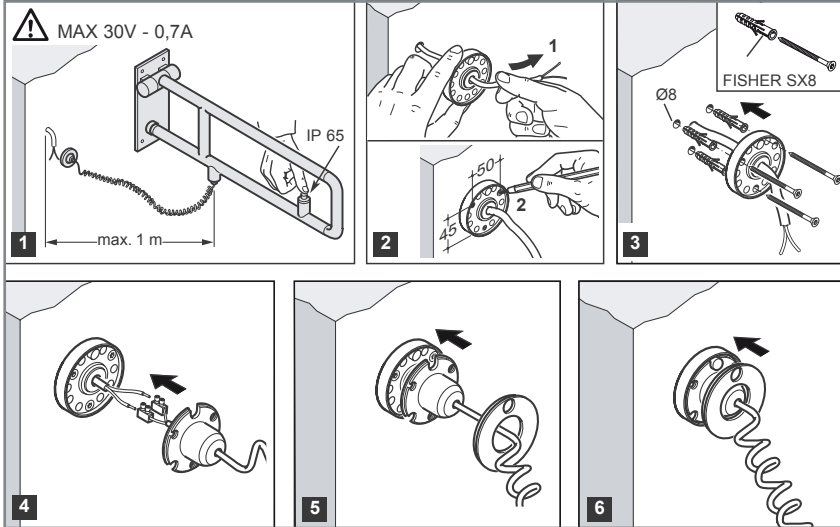
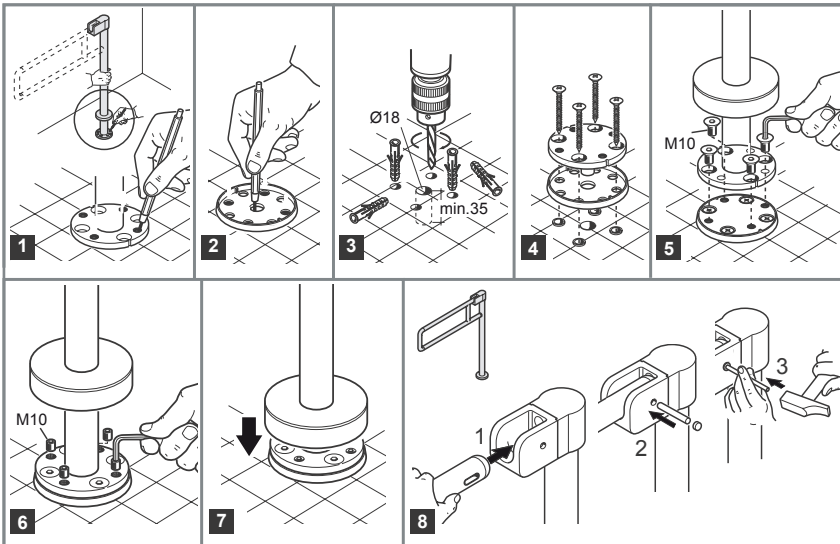
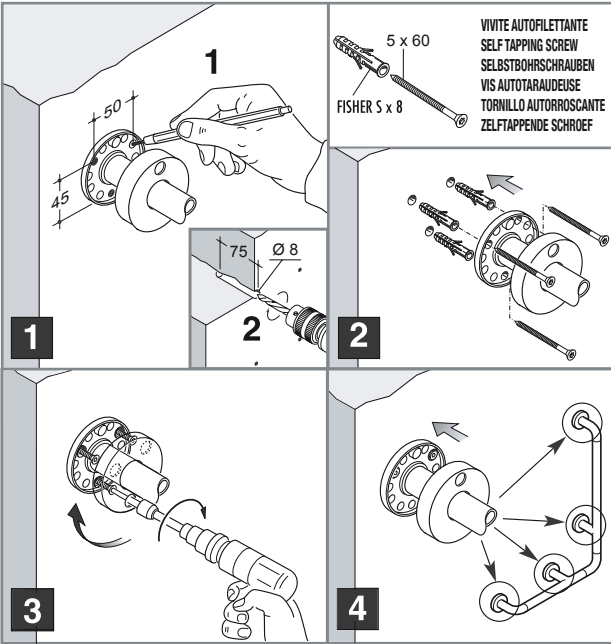
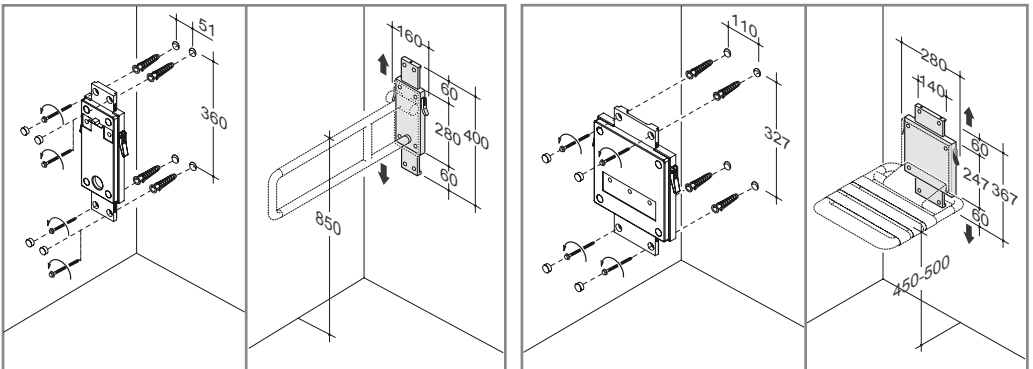
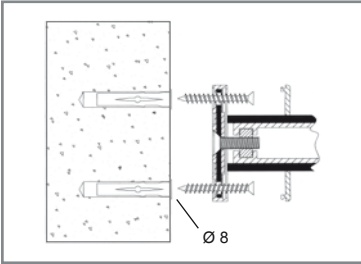
Drill the wall or the ceiling in correspondence to the three chosen rose holes. Then fix the rose by means of the plugs and the screws supplied with the fixing set. Finally fix the click cap.

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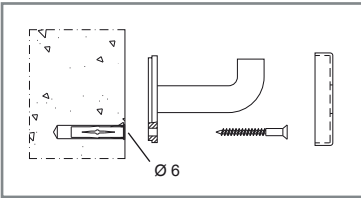
Drill the wall in correspondence to the rose holes. Then fix the rose by means of the plugs and the screws supplied with the fixing set. Finally fix the click cap.

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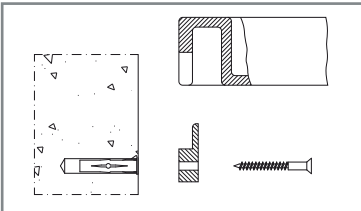
Mark location of the holes. Drill holes according to the measurements as described below. Fix on the wall the two nylon supports by means of plugs and screws. Insert the bathroom shelf on the wall supports.

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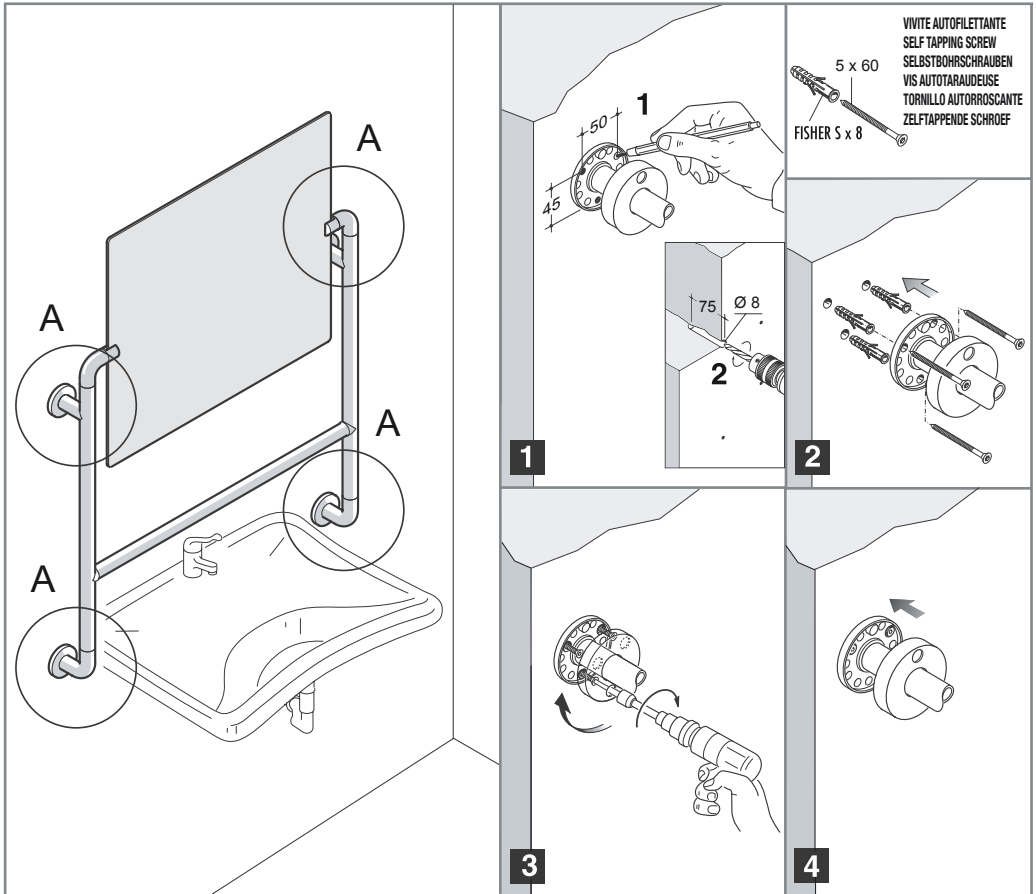
Equipped structure with mobile mirror, vertical supports and horizontal support

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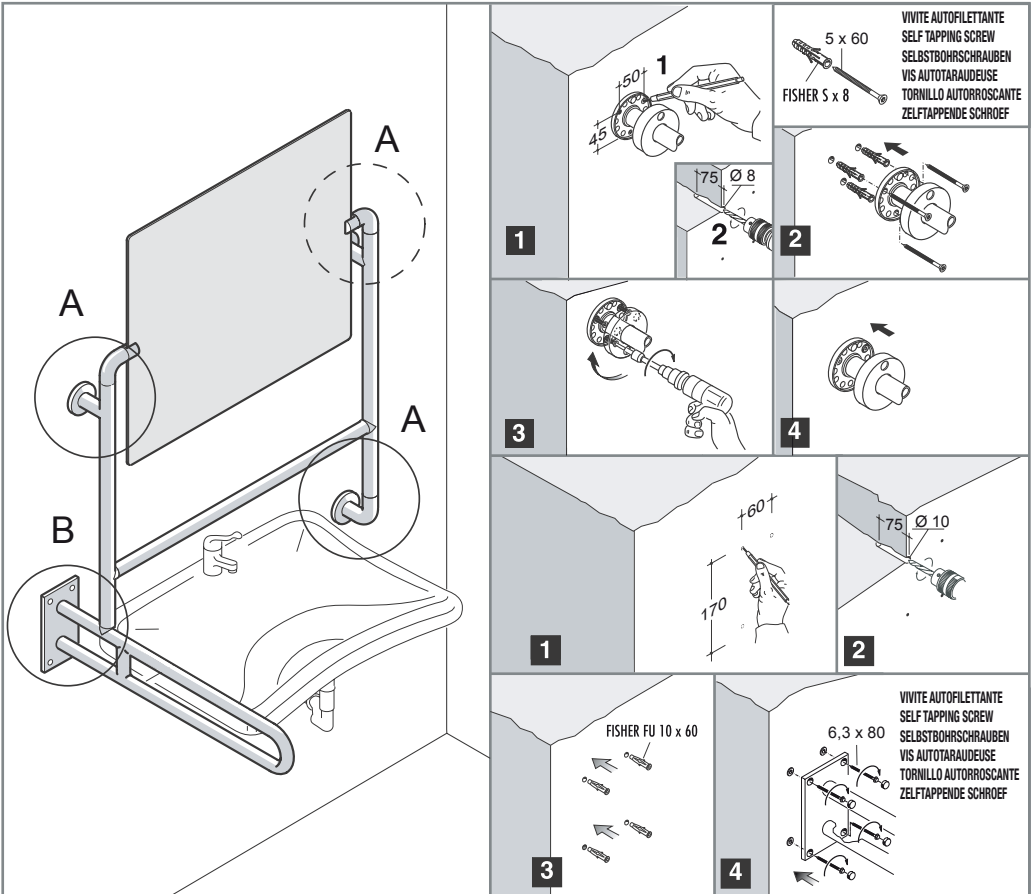
Equipped structure with mobile mirror, vertical supports, horizontal support and lateral support

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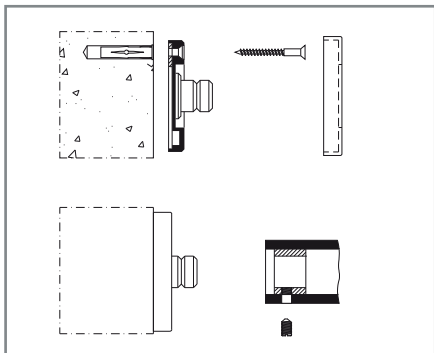
Mark location of the holes. Drill holes according to the three holes of the rose as described below. Fix the rose by means of plugs and screws according to the centres previously marked on the wall. Fix the nylon cap on the rose. Insert the side mirror holders into the wall supports, fix the mirror with the wall supports on the brass bolts of the wall roses and secure them with headless pins supplied with the fixing set.

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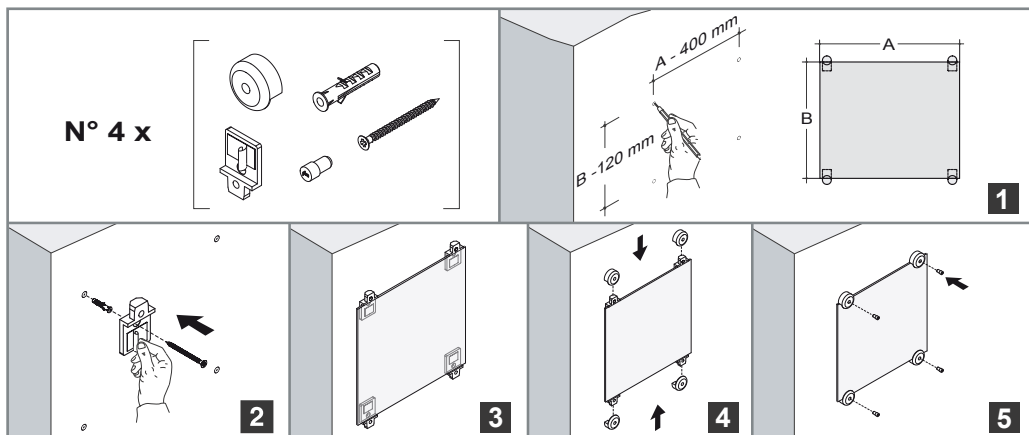
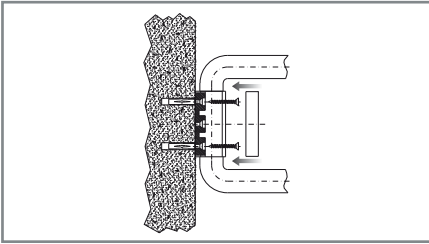
Mark location of the holes. Drill holes according to the measurements as described below. Fix the plate by means of plugs and screws. Finally fix the click cap.

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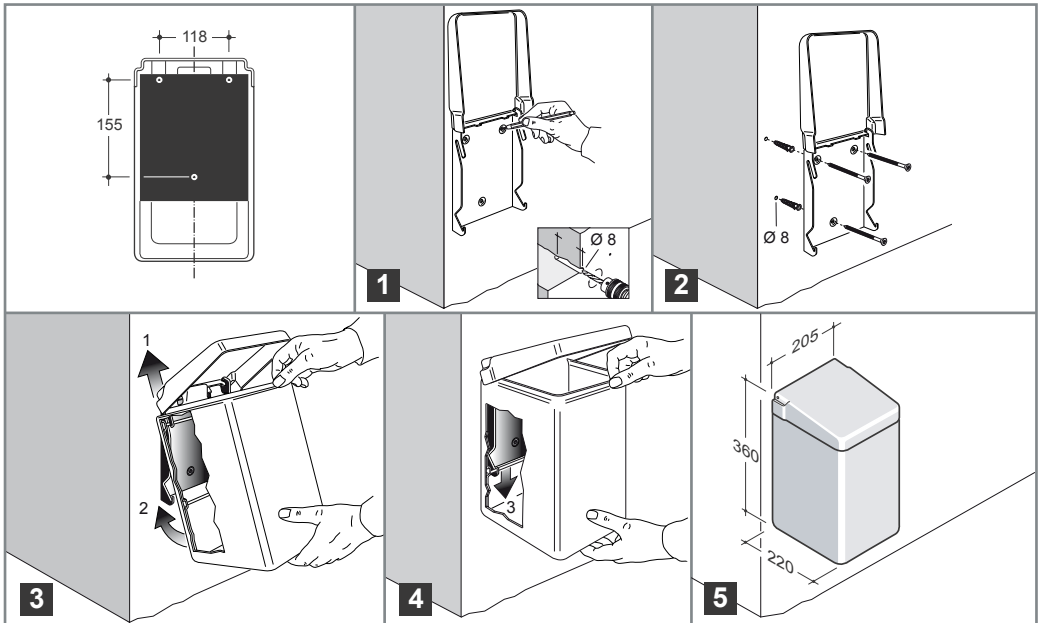
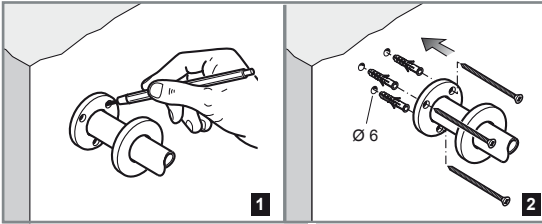
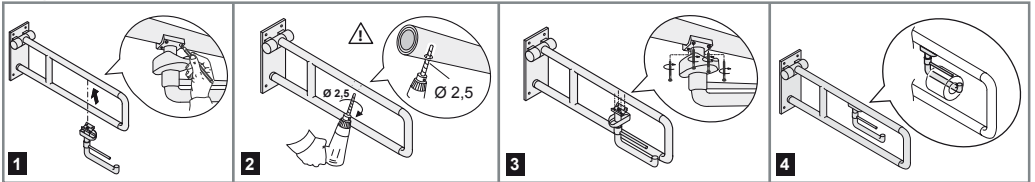
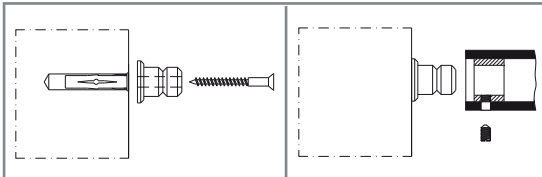
Mark location of the holes. Drill holes according to the following centres and measurements. Drive in the plugs, screw in the bolt. Insert the bathroom accessory and secure it with the headless pin supplied with the fixing set.

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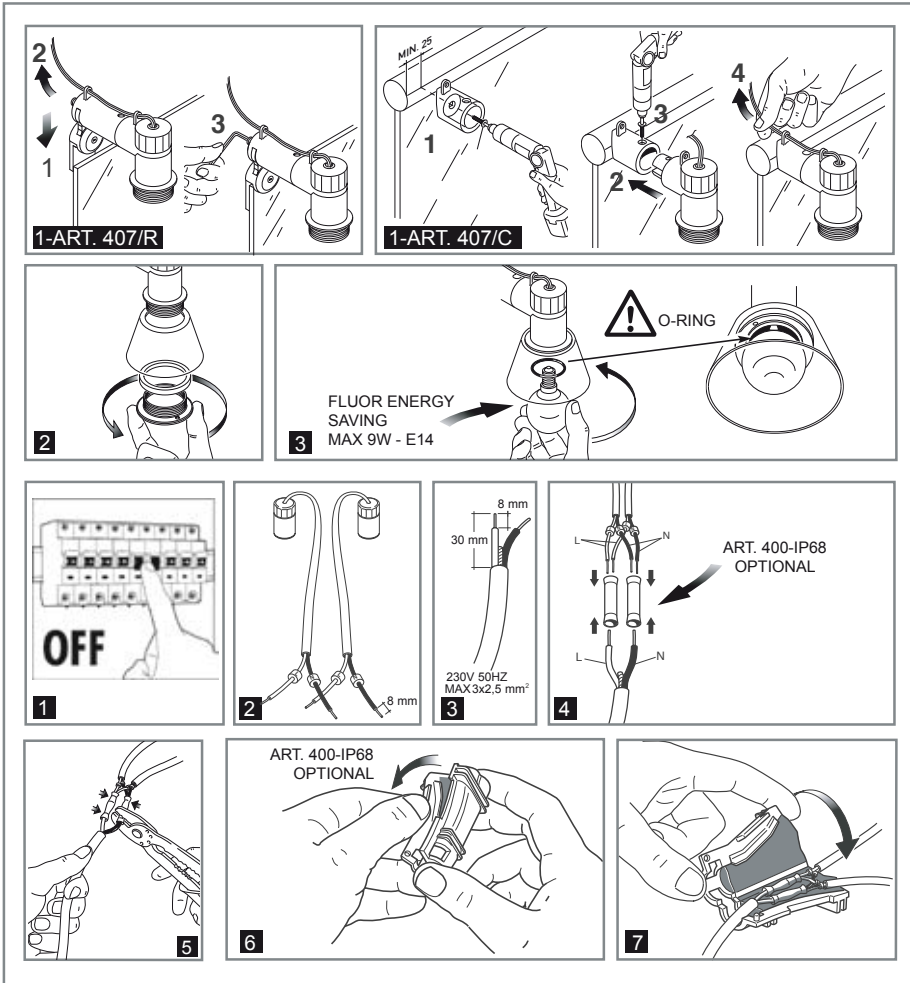
Terminal block (ART. 400-IP 68) non included. Installation may require advice from a qualified person (CEI EN 60598-1:2001:001 3.3. 101).

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Pict. 1 left safety support - Disassemble the support like pict. 2 - Wheel the bends of 180° like pict. 3 - Assemble all the parts like pict. 4. Now the support is right.

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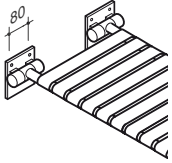
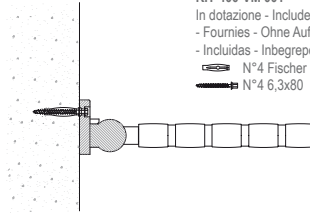
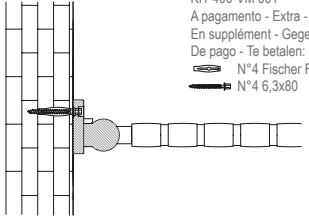
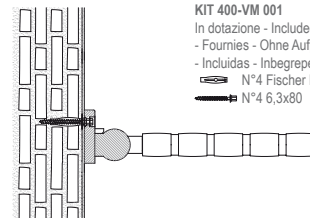
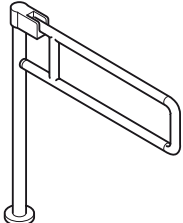
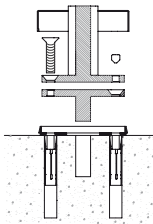
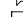
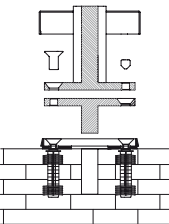
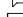
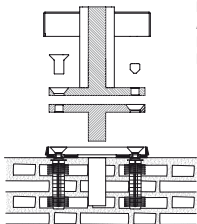
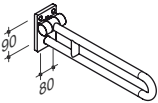
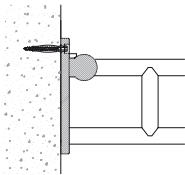
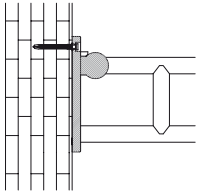
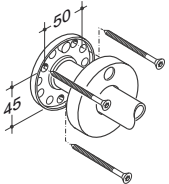
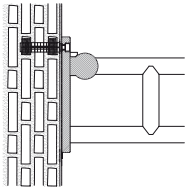
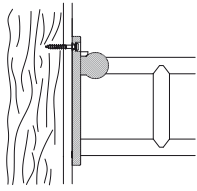
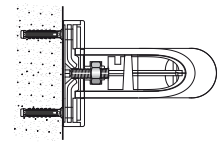
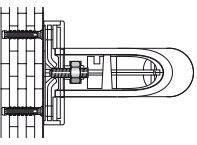
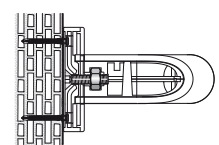
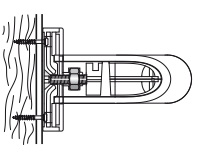
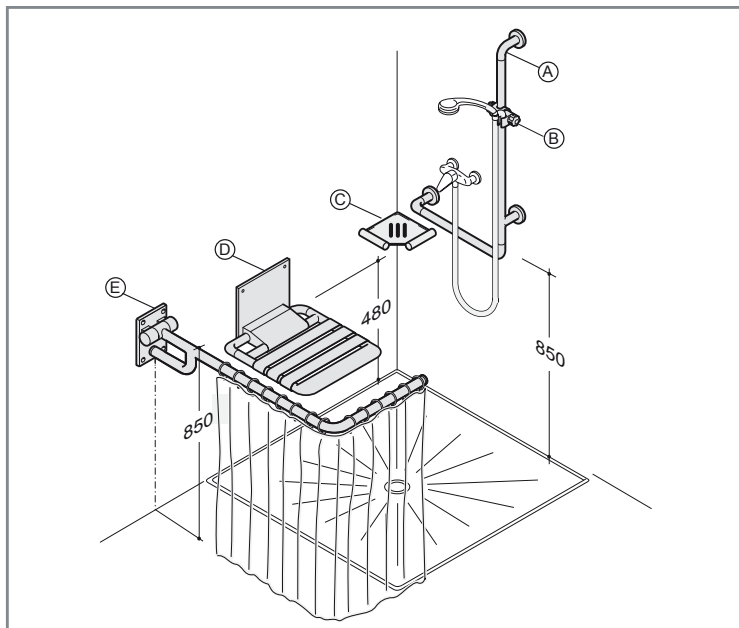
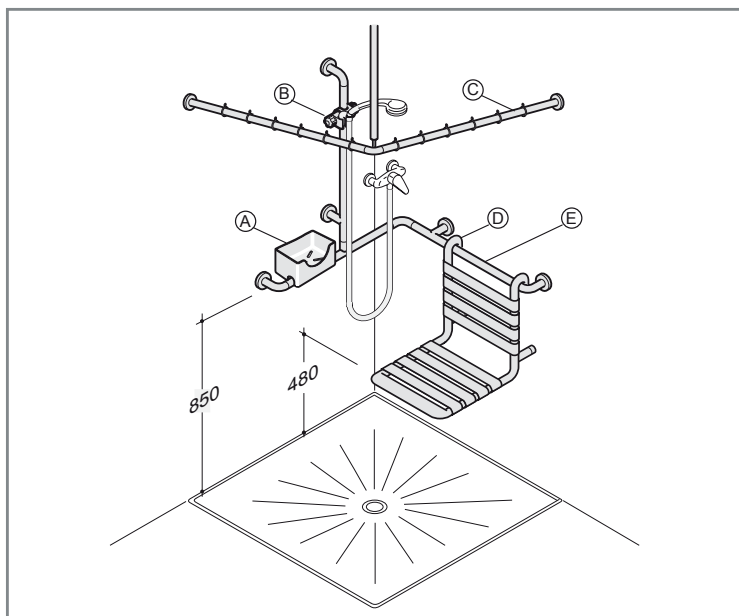
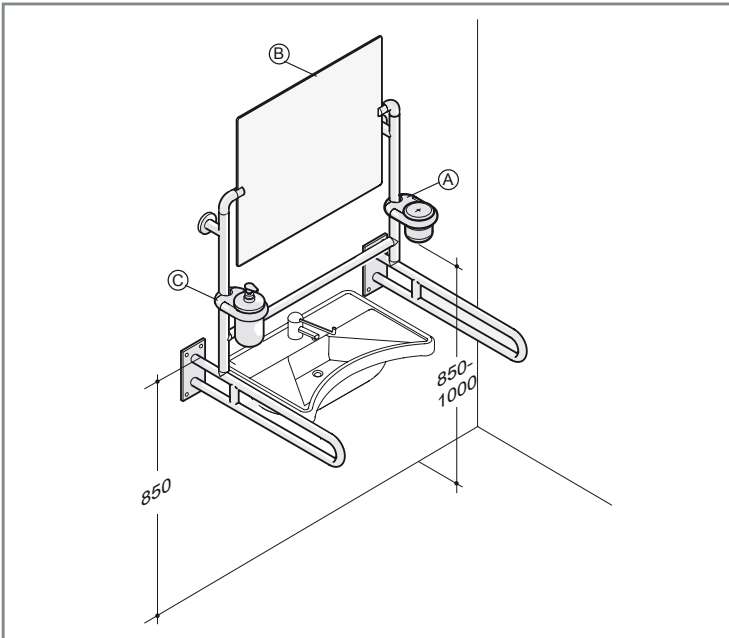
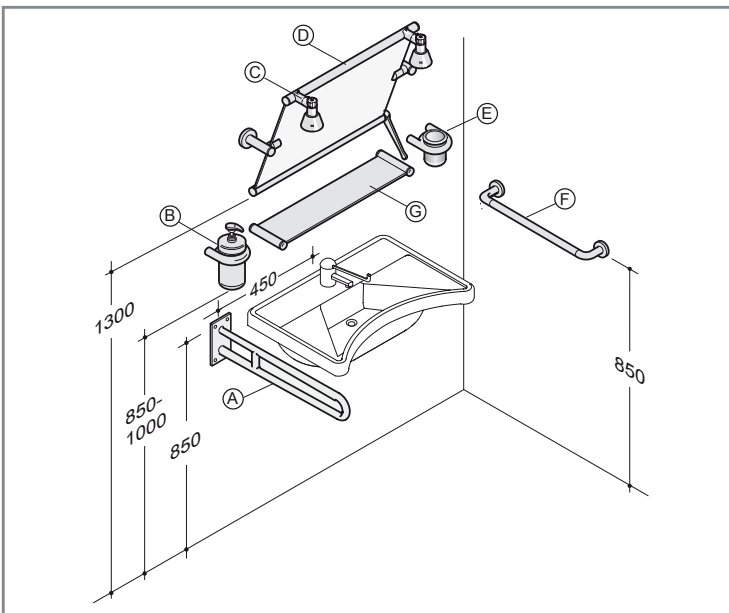
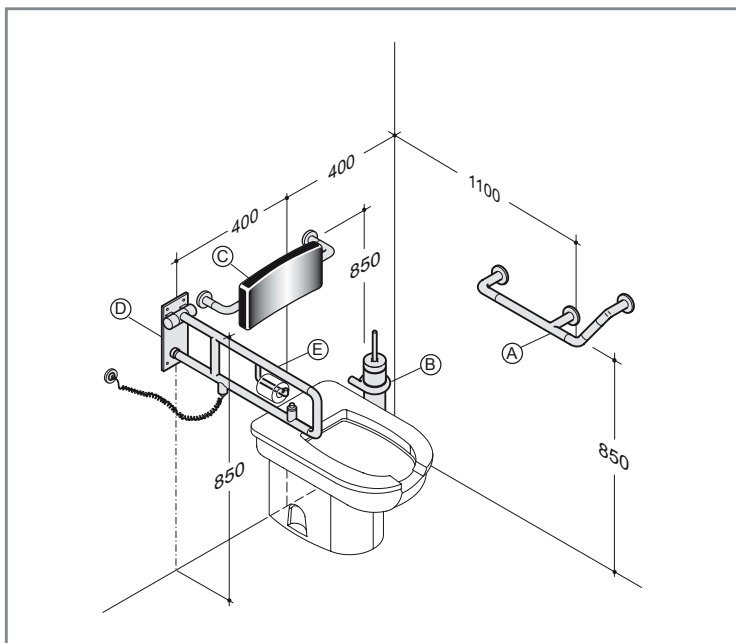
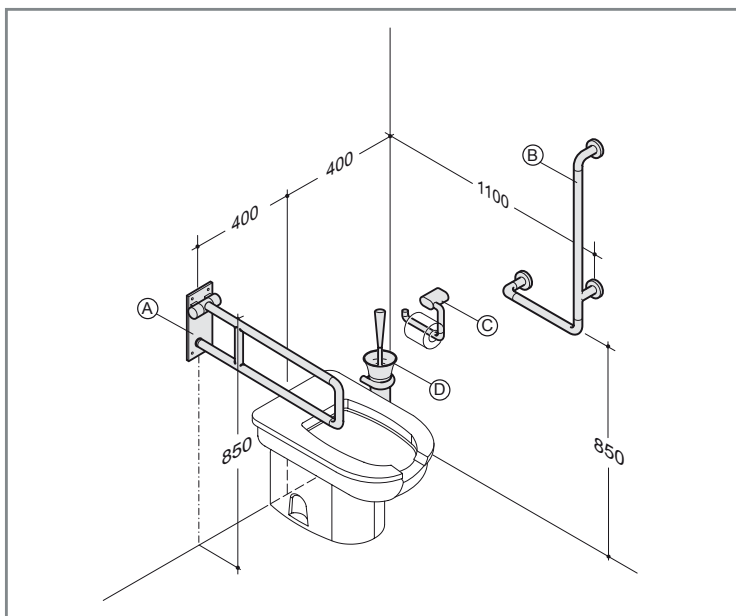
The Nylon used is a thermoplastic resin (Polyamide 6) which guarantees:

- high mechanical resistance
- shock resistance
- resistance to wear and abrasion
- antistatic behaviour
- excellent stability with chemical agents such as hydrocarbons, oils and greases.

Furthermore, the resin used is physiologically innocuous as proved by the cytotoxicity test carried out by a specialised German laboratory HYGECEN on all of our colourings.

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Client: ...  
 Product: ...  
 Reference: ...

Conformity Test to EN ISO 10993-5  
 01.06.2020

Sample number: 01-1000-01  
 Delivery date: 01.06.2020  
 Analysis: ...  
 Evaluation: ...  
 Manufacturer: ...  
 Test method: ...

Test date period: 01.06.2020 - 01.06.2020  
 Test conditions: ...

EN ISO 10993 page 1 of 1

**Description of the method**

**Extraction conditions:** ...

**Cell culture:** ...

**Exposition:** ...

**Test cell count the wells:** ...

**Evaluation:** ...

**Results**

**Figure 1: Test result of the culture**

**Table 1: Description available substances**

Substance	Concentration	Cell Count	Cell Count (%)
Control	0.01%	1000	100%
Sample 1	0.01%	950	95%
Sample 2	0.01%	900	90%
Sample 3	0.01%	850	85%
Sample 4	0.01%	800	80%

Test result: ...

**Conclusion**

After testing the Conformity of the '...' according to the EN ISO 10993-5: 2008-11 - our best report of 01.06.2020 - we give the following statement:

From the tested material only minimal cytotoxic effects were observed at 27°C. The amount of the test material released into the cell growth is 0.01% (w/v) (w/v). This is not statistically significant (p > 0.05) (Tab. 1).

Using the test material as described by the manufacturer no cytotoxic effects should be expected.

**SPECIFICHE TECNICHE E CERTIFICAZIONI - TECHNICAL EXPLANATIONS AND CERTIFICATIONS - EXPLICATIONS TECHNIQUES ET CERTIFICATIONS - TECHNISCHE SPEZIFIKATIONEN UND ZERTIFIKATE - ESPECIFICACIONES TECNICAS Y CERTIFICACIONES - TECHNISCHE SPECIFICATIES EN CERTIFICATIES**

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A special pressure die-cast aluminium alloy is used for the core of the parts coated in thick Nylon to guarantee both excellent mechanical strength and extraordinary lightweight, along with excellent resistance to corrosion.

In particular for the straight tube, the choice of extruded aluminium as the core of the support for the Nylon sleeve guarantees lighter weight and higher resistance to corrosion with respect to traditional steel. Furthermore, the excellent workability of aluminium makes it possible to modify the length of the handrail during installation without special equipment or specialised personnel.

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The products of the Programma 400-NY CE marked are in conformity with the requirements of the directive 93/42/CEE for medical devices and to the requirements of the directive BT 73/23/CEE (modified by the directive 93/68/CEE) for low-tension electrical apparatus.

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The products of the 400-NY programme have been tested and certified by TÜV Product Service and have the TÜV Production Monitored/Safety Test mark issued after successful testing, production verification and assessment of the organisation. In particular, the two last aspects are kept under control by the certification board through detailed annual inspections to assess the entire production process and internal quality audits.

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The lamps for PBA mirrors have been tested in a specialised laboratory and have IP 54 protection as well as the CE declaration of conformity.

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PRODUCT: **LAMPS FOR MIRRORS**

ITEMS: **LAMPS FOR MIRRORS**

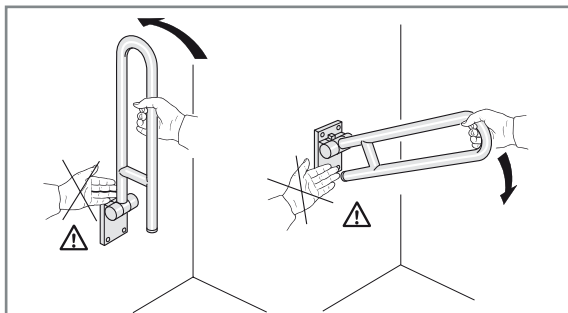
I, the undersigned, hereby declare that the lighting apparatus specified above, has been designed and manufactured in compliance with the directive 93/72/EEC, modified from directive 93/88/EEC in Italy by the legislative decree 430 of 12/21/99 and that the specification CIE EN 60598-1 has been applied.  
 The product is signed with the CE mark.

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
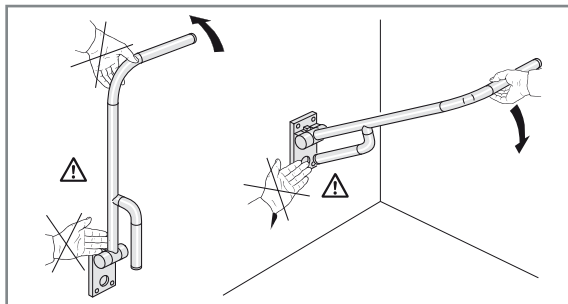
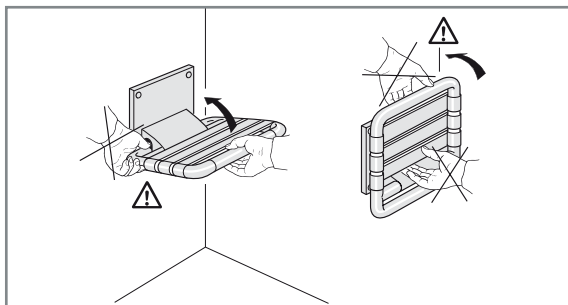
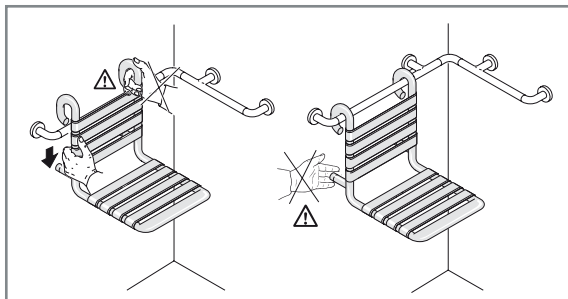
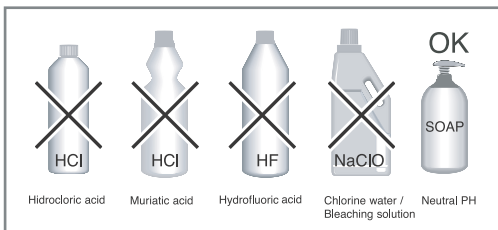
**CLEANING OF NYLON PARTS:** please DO NOT use soaps or detergents containing chlorine, chloroform, chloride water, acids, bromine, cresol, carboic acid, formaldehyde, formalin or potassium permanganate regardless of percentage content. PBA declines all responsibility for damage caused by use of above mentioned aggressive detergents. The nylon parts feature excellent stability to hydrocarbons, oil fats. They also have good stability to halogenated hydrocarbons such as carbon tetrachloride, frigen and freon. According to the above, we recommend disinfectants ba alcohols and also ammoniac and phosphor compounds.

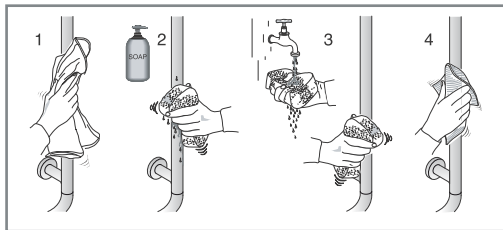**CLEANING STAINLESS STEEL PARTS:** these can be sanitized with soap and water or neutral detergents. First, delicately remove any dust on the surfaces, then wash the stainless steel parts, and finally rinse and if necessary dry the part. Wash the part in the direction of the satin finish to avoid scratching the surfaces. Never use products that release chlorine ions; for example, hydrochloric acid or similar solutions. Furthermore, avoid using abrasive sponges or steel wool, as these will contaminate the surfaces of the material. Finally, stubborn deposits on the surfaces of the part must be removed using a wood spatula (not a steel one), to avoid scratching and contamination.

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The following installation procedures refer to the fixing of disabled hardware to concrete. For other types of building construction, refer to the table of fixing material. For articles 441-SS/P, 445-SS, 447-SS/B, 435-SS, 480-SS, 481-SS, 483-SS, 490-SS, 492-SS we recommend the use of assembly template, art. 499, to set out the fixing hole centres.

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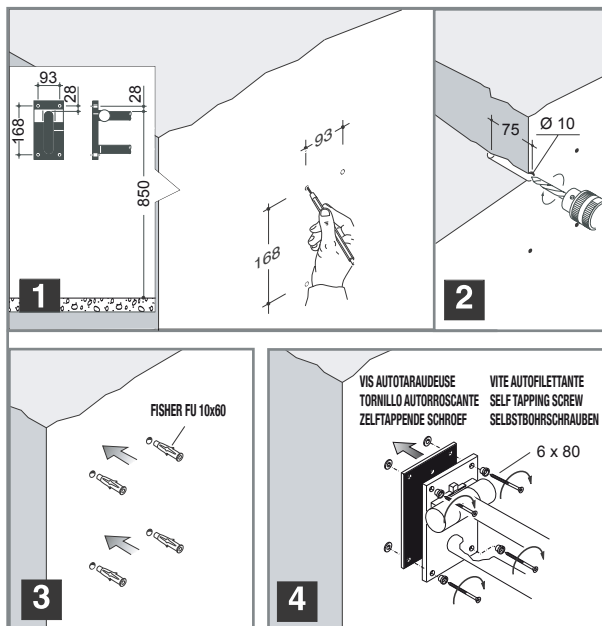
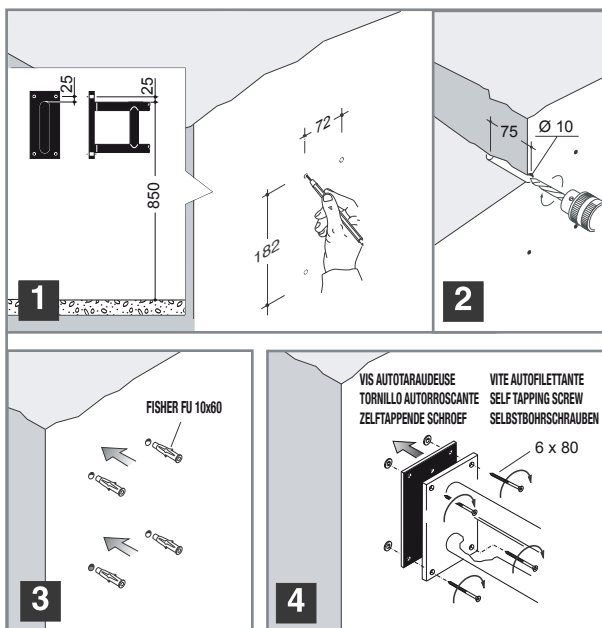
Lift-up safety support and lift-up support with shower curtain on plate 125x200 mm.

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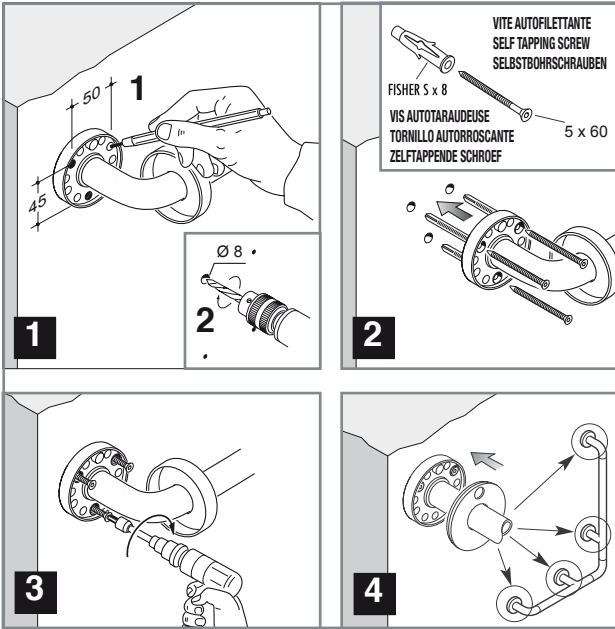
Ausili su rosetta  $\varnothing$  77 mm

Disabled accessories on  $\varnothing$  77 mm washer

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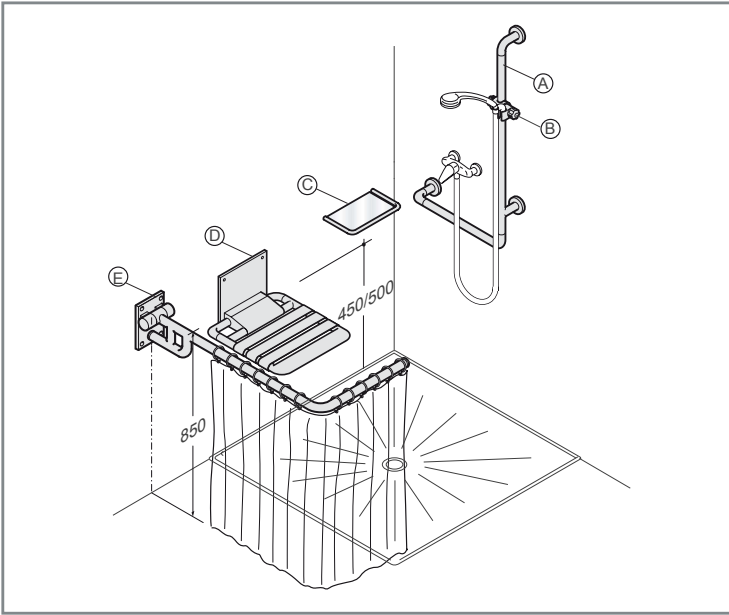
NOTE: for fixing to other materials contact the PBA technical office.

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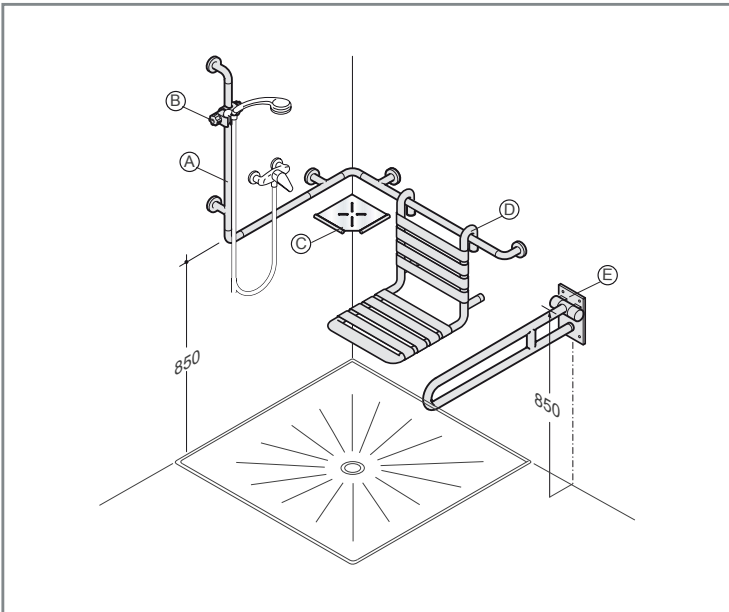
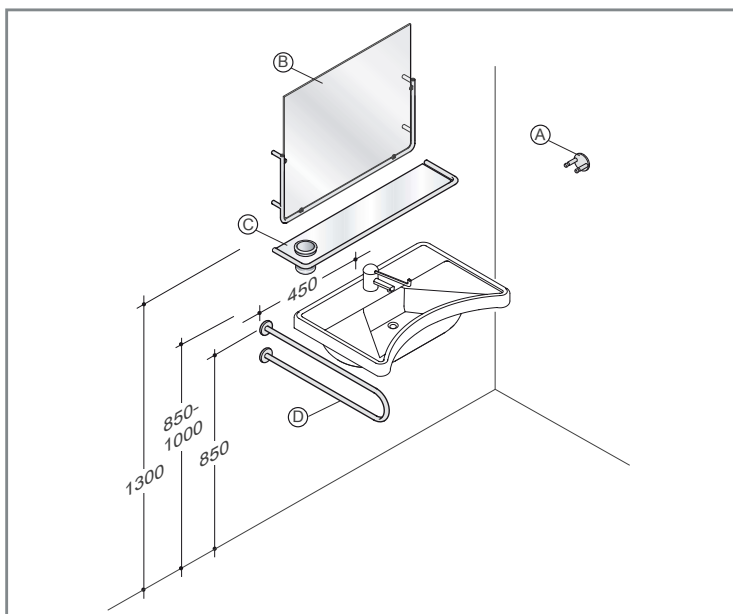
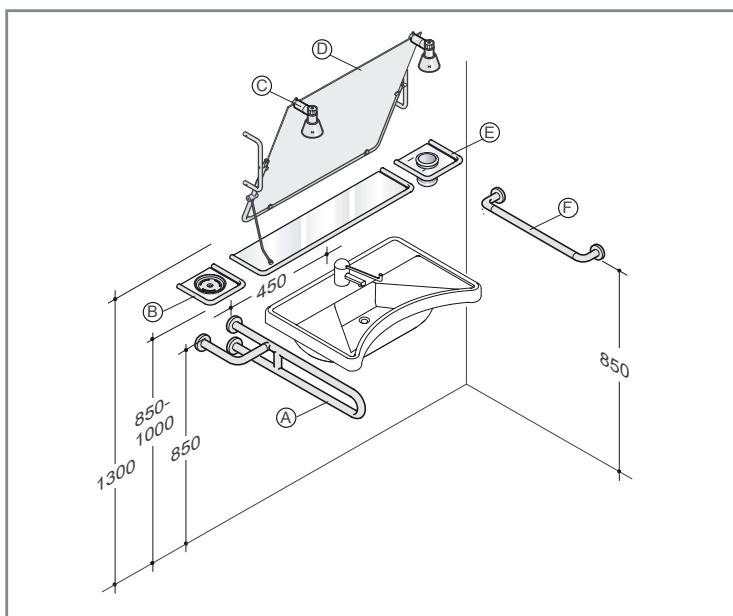
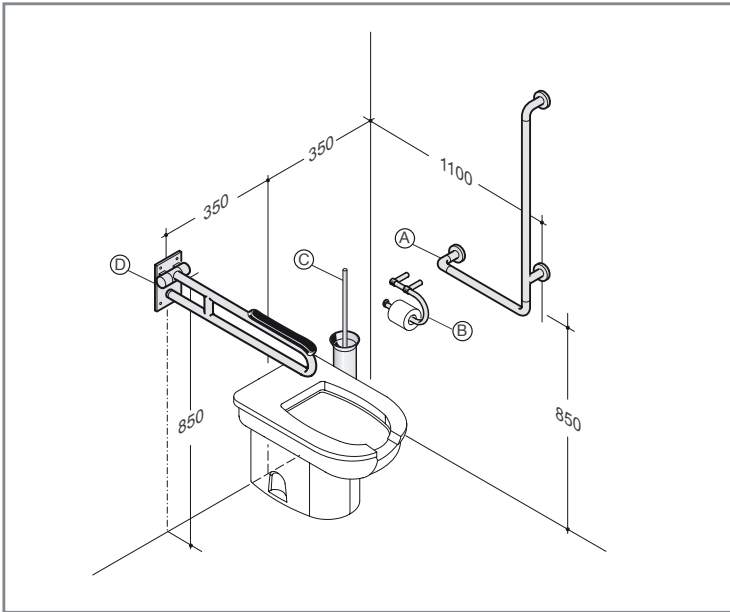
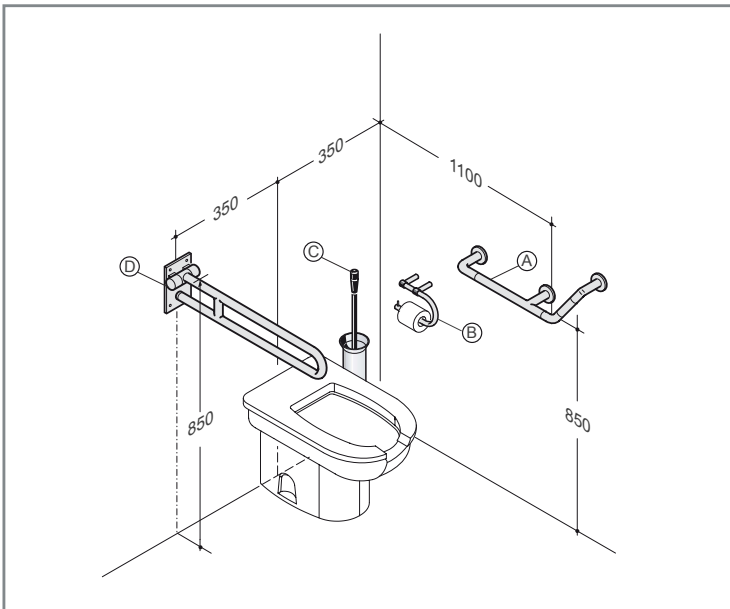
The AISI 316L stainless steel (1) steel used by pba, meets the requirements of UNI EN 1670, the ultimate quality standard for ensuring resistance to corrosion. Pba has been awarded a certificate for successfully salt spray testing the stainless steel in accordance with UNI ISO 9227.

The 400-SS programme articles are available with satin or polished finish.

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The products of the Programma 400-SS CE marked are in conformity with the requirements of the directive 93/42/CEE for medical devices and to the requirements of the directive BT 73/23/CEE (modified by the directive 93/68/CEE) for low-tension electrical apparatus. Our disabled accessories have class 1 medical classification.

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The products of the 400-SS programme have been tested and certified by TÜV Product Service and have the TÜV Production Monitored/Safety Tested Test mark issued after successful testing, production verification and assessment of the organisation. In particular, the two last aspects are kept under control by the certification board through detailed annual inspections to assess the entire production process and internal quality audits.

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PR 4.1.1  
Via Enrico Fermi s.r.l.  
36060 Tesse San Biagio (VI)  
Italy

with production facility:  
29013

is authorized to issue the following products with the "TUV Mark P"

Product: **Sanitary items for disabled persons**  
Sanitary Aids for Disabled Persons

Model: **Shower call and seats**  
Series 400-SS

Parameters: see enclosure

The compliance of the above-mentioned product with the following essential requirements was tested on a voluntary basis.  
EN 12461:2009

Report No.: 7902008

Released with the above certificate number by the certification body of TÜV PRODUCT SERVICE GMBH

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Remark:  
All products are made of stainless steel (V4A)

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PR 4.1.1  
Via Enrico Fermi s.r.l.  
36060 Tesse San Biagio (VI)  
Italy

with production facility:  
29013

is authorized to issue the following products with the "TUV Mark E21"

Product: **Sanitary item**

Model: **Shower call and seat**  
model 420-SS

Parameters: nominal load: 400 x 100 mm  
max. height: 100 mm  
max. weight: 20 kg

The compliance of the above-mentioned product with the following essential requirements was tested on a voluntary basis.  
EN 21400:1979

Report No.: 7902492

Released with the above certificate number by the certification body of TÜV PRODUCT SERVICE GMBH

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PR 4.1.1  
Via Enrico Fermi s.r.l.  
36060 Tesse San Biagio (VI)  
Italy

with the production facility:  
29013

is authorized to issue the following product with the "TUV Mark P"

Product: **Sanitary Aids for Disabled Persons**

Model: **Shower seats**  
Series 400-SS

Parameters: Dimensions in (mm):  
Type: 400-SS-A, 400-SS-B  
Profile: 6 R x 2, Ø 32 x 12  
Dimensions: 400x600, 400x800, height 100/150/200  
Material: Stainless steel A2 (EN 19 2)  
Normal load: 150 kg

Remark: Seat to use only together with Series 400-SS

The compliance of the above-mentioned product with the following essential requirements was tested on a voluntary basis.  
EN 12461:2009

Report No.: MEC 0228

Released with the above certificate number by the Certification body of TÜV PRODUCT SERVICE GMBH

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**ATTESTATION**

**TÜV**  
PRODUCT SERVICE

Production facility inspection on the basis of the agreement in the testing and certification rules performed on July 28, 2005.

Location holder:  
P.R.A. S.R.L.  
Via Enrico Fermi, 1  
36060 TESSE SAN BIAGIO (VI)  
ITALIA

This facility manufactures products certified by TÜV PRODUCT SERVICE. The requirements to a healthy and homogeneous fabrication is stated in the test and certification rules as well. The signed and documented quality assurance procedures were found suitable.

Production facility:  
P.R.A. S.R.L.  
Via Enrico Fermi, 1  
36060 TESSE SAN BIAGIO (VI)  
ITALIA

Equipment tested:  
Sanitary items for disabled persons / Door and window frames

The production facility inspection was conducted according to the instructions of TÜV PRODUCT SERVICE. This certificate is valid for 12 months, starting with the date of inspection.

Scannopin, 28/08/2005

hereby certify:  
TÜV PRODUCT SERVICE  
Franco L. CA  
Product Service

**TÜV**  
PRODUCT SERVICE

Production monitored  
Safety tested

PBA S.p.A.  
Via Enrico Fermi, 1 - I-36056 Tezze sul Brenta (VI)  
Tel. +39 0424 5451 - Fax +39 0424 54522  
info@pba.it - www.pba.it

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PBA Deutschland GmbH